## Dimension® clinical chemistry system

### Flex® reagent cartridge

**ALC**

See shaded sections: Updated information from 2006-08 version.


**Issue Date 2008-08-19**

#### Ethyl Alcohol

**Intended Use:** The ALC method used on the Dimension® clinical chemistry system is an *in vitro* diagnostic test intended to measure ethyl alcohol in human serum and supernatants from precipitated whole blood and to qualitatively detect ethyl alcohol in urine. Ethyl alcohol test results may be used in the diagnosis and treatment of alcohol intoxication and poisoning.

**Summary:** The ethyl alcohol (ALC) method is a modification of the alcohol dehydrogenase (ADH) enzymatic procedure originally described by Bonnicksen and Lundgren<sup>1</sup> and later presented as a general procedure by Bergmeyer.<sup>2</sup> This method uses Tris (hydroxymethyl) aminomethane buffer as an aldehyde trapping agent.

**Principles of Procedure:** Alcohol dehydrogenase (ADH) catalyzes the oxidation of ethanol to acetaldehyde, with the simultaneous reduction of nicotinamide adenine dinucleotide (NAD). An alkaline pH and an aldehyde trapping agent force the reaction to one mole of NADH for each mole of alcohol present. The absorbance due to NADH (and thus the alcohol concentration) is determined using a two-filter (340 – 383 nm) endpoint technique.

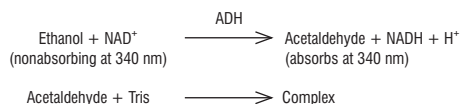#### Reagents

Wells <sup>a</sup>	Form	Ingredient	Concentration <sup>b</sup>	Source
1, 2	Liquid	ADH	15 U	Yeast
3, 4, 5, 6	Tablet	NAD	4.9 µmol	
7, 8	Liquid	Tris buffer	0.95 mol	

a. Wells are numbered consecutively from the wide end of the cartridge.  
b. Nominal value per test at manufacture.

#### Risk and Safety:

	Irritant.
	R36/37/38: Irritating to eyes, respiratory system and skin.
	S24: Avoid contact with skin.
	S36/37: Wear suitable protective clothing and gloves.

Safety data sheets (MSDS/SDS) available on [www.siemens.com/diagnostics](http://www.siemens.com/diagnostics)

**Precautions:** Used cuvettes contain human body fluids; Handle with appropriate care to avoid skin contact and ingestion.

For *in vitro* diagnostic use

**Reagent Preparation:** Hydrating, mixing and diluting are automatically performed by the instrument.

**Store at:** 2 – 8 °C

**Expiration:** Refer to carton for expiration date of individual unopened reagent cartridges. Sealed or unhydrated cartridge wells on the instrument are stable for 30 days.

**Open Well Stability:** 5 days for wells 1 – 8

**Specimen Collection and Handling:** Use non-alcohol germicidal solution to cleanse the skin. If reusable containers, syringes, and needles are employed, they must not be cleaned or stored with alcohol or other volatile solvents. The specimen tube should be completely filled and stored under refrigeration until analyzed.<sup>3,4,5</sup> To minimize the losses of alcohol in a sample due to evaporation, open and process samples in STAT mode. Urine specimens should be kept in sealed containers at 4 °C until analysis. For prolonged or unrefrigerated storage, add 50 mg NaF as a preservative.<sup>6</sup>

Lithium heparin and potassium oxalate/sodium fluoride anticoagulants do not interfere with ALC method at concentrations normally found in blood collection tubes.

Whole blood preserved with 4.3 mg/mL [1.02 mmol/L] NaF and visibly hemolyzed serum may be pretreated by precipitation with 6% (w/v) TCA prior to analysis with the ALC method.

Follow the instructions provided with your specimen collection device for use and processing.<sup>7</sup>

#### Precipitation Procedure

1. Prepare a working solution of 6% (w/v) trichloroacetic acid (TCA) each day of use by making a 1: 5 dilution of stock 30% (w/v) TCA solution.<sup>c</sup>
2. Gently invert whole blood samples ten times.
3. Using volumetric pipets, add 1.0 mL of whole blood to 2.0 mL of working 6% (w/v) TCA.
4. Invert mixture ten times. Centrifuge for 5 minutes at 200 x g. Analyze supernatant on the Dimension® system immediately.
5. Multiply readout by dilution factor of three (3) to obtain "whole blood" ethyl alcohol result on the Dimension® system.

c. TCA solutions of less than 30% are considered unstable.<sup>8</sup>

#### Procedure

##### Materials Provided

ALC Flex® reagent cartridge, Cat. No. DF18

##### Materials Required But Not Provided

ALC Calibrator, Cat. No. DC37A  
Quality Control Materials

#### Test Steps

Sampling,<sup>d</sup> reagent delivery, mixing, processing, and printing of results are automatically performed by the Dimension® system. For details of this processing, refer to your Dimension® Operator's Guide.

d. The sample container (if not a primary tube) must contain sufficient quantity to accommodate the sample volume plus dead volume. Precise container filling is not required.

#### Test Conditions

Sample Size	3 µL
Reagent 1 Volume	190 µL
Reagent 2 Volume	90 µL
Reagent 3 Volume	25 µL
Diluent Volume	192 µL
Temperature	37 °C
Wavelength	340 and 383 nm
Type of Measurement	bichromatic, end point

#### Calibration

Assay range	0 – 300 mg/dL [0.0 – 65.0 mmol/L] <sup>9</sup>
Calibration Material	Alcohol Calibrator (Cat. No. DC37A). Opened ampules should be processed immediately due to evaporation.
Calibration Scheme	3 levels, n = 3
Units	mg/dL [mmol/L] (mg/dL x 0.217) = [mmol/L]
Typical Calibration Levels	0, 100, 300 mg/dL [0, 22.0, 65.0 mmol/L]
Calibration Frequency	Every 3 months for any one lot
A new calibration is required	<ul style="list-style-type: none"> <li>• For each lot of Flex® reagent cartridges</li> <li>• After major maintenance or service, if indicated by quality control results</li> <li>• As indicated in laboratory quality control procedures</li> <li>• When required by government regulations</li> </ul>
Assigned Coefficients	C <sub>0</sub> -2.600 C <sub>1</sub> 0.323

e. Système International d'Unités [SI Units] are in brackets.

#### Quality Control

At least once daily run solutions at two levels of a quality control material with known concentrations of ethyl alcohol. For further details, refer to your Dimension® Operator's Guide. The result obtained should fall within limits defined by the day-to-day variability of the system as measured in the user's laboratory. If the results fall outside the laboratory's acceptable limits, refer to your Dimension® Operator's Guide.

#### Urine Ethyl Alcohol Screen Method

At least once daily analyze a POSITIVE and a NEGATIVE control using a suitable control material. The results obtained should fall within acceptable limits defined by the day-to-day variability of the system as measured in the user's laboratory.

**Results:** The instrument automatically calculates and prints the concentration of ALC in mg/dL [mmol/L] using the calculation scheme illustrated in your Dimension® Operator's Guide. These results should be reported if the specimen is serum or supernatant from precipitated whole blood. Urinary ALC results should be reported as NEGATIVE for results less than 50.0 mg/dL [11.0 mmol/L] or POSITIVE for results greater than or equal to 50.0 mg/dL [11.0 mmol/L]. POSITIVE results should be confirmed by an alternate method such as gas chromatography.<sup>9</sup> The physiological and psychological effects of ethanol do not necessarily correlate with urinary concentrations. The presence of ethanol in urine is only an indication of recent ingestion. Determinations on serum or whole blood are needed to assess the effect.<sup>10,11</sup>

**Results of this test should always be interpreted in conjunction with the patient's medical history, clinical presentation and other findings.**

**Note:** Check state and local laws to determine whether the measurement of ethyl alcohol using this method can be accepted for establishing legal blood alcohol levels.

**Analytical Measurement Range (AMR): 0 – 300 mg/dL [0.0 – 65.0 mmol/L]**

This is the range of analyte values that can be measured directly from the specimen without any dilution or pretreatment that is not part of the usual analytical process and is equivalent to the assay range.

Samples with results in excess of 300 mg/dL [65.0 mmol/L] should be repeated on dilution.

**Manual Dilutions:** Make appropriate dilutions with Reagent grade water to obtain result within the assay range. Enter dilution factor. Reassay. Resulting readout is corrected for dilution.

**Autodilution (AD):** Refer to your Dimension® Operator's Guide.

Results less <0.0 mg/dL [0 mmol/L]

To verify the result is less than 0, dilute 1 part of sample with 1 part of a standard or control containing a known concentration of alcohol. Reassay. If the result of this mixture is half the known value of the standard or control, report the value of the sample as 0 mg/dL [0 mmol/L]. Report specimen type analyzed (i.e. serum, whole blood or urine).

#### Limitations of Procedure

The instrument reporting system contains error messages to warn the operator of specific malfunctions. Any report slip containing such error messages should be held for follow-up. Refer to your Dimension® Operator's Guide for troubleshooting.

A system malfunction may exist if the following 5-test precision is observed:

Concentration	SD
100 mg/dL [22.0 mmol/L]	> 3 mg/dL [0.7 mmol/L]
300 mg/dL [65.0 mmol/L]	> 6 mg/dL [1.3 mmol/L]

### Interfering Substances

Isopropyl alcohol of 51 mg/dL [8.5 mmol/L] increases the ethyl alcohol by 11 mg/dL [2.4 mmol/L] at an ethyl alcohol concentration of 100 mg/dL [22.0 mmol/L]; Isopropyl alcohol of 254 mg/dL [42.3 mmol/L] increases the ethyl alcohol by 44 mg/dL [9.6 mmol/L] at an ethyl alcohol concentration of 100 mg/dL [22.0 mmol/L].

At ethyl alcohol concentration of 100 mg/dL [22 mmol/L], butanol at 250 mg/dL increases the ALC result by 26.5% and n-propanol at 500 mg/dL increases the ALC result by 57.7%.

Bilirubin (unconjugated) of 40 mg/dL [384 µmol/L] will decrease an ALC result at 93 mg/dL [20.4 mmol/L] by 11%.

Lipemia (Intralipid®) of 3000 mg/dL [33.9 mmol/L] will increase an ALC result of 92 mg/dL [20.4 mmol/L] by 16%.

\*Intralipid® is a registered trademark of Fresenius Kabi AG, Bad Homburg, Germany.

**Therapeutic Range:** The pharmacological response to blood alcohol levels may vary from individual to individual. The fatal concentration has been reported to be greater than 400 mg/dL [87.0 mmol/L].<sup>12</sup>

### Specific Performance Characteristics<sup>1</sup>

Material	Mean mg/dL [mmol/L]	Precision <sup>a</sup>		Standard Deviation (% CV)	
		Within-run		Between-day	
		mg/dL [mmol/L]		mg/dL [mmol/L]	
Whole blood Supernatant	30.3 [6.6]	0.71 [0.2]	(2.4)	1.68 [0.4]	(5.71)
Plasma Pool	93.0 [20.2]	1.80 [0.4]	(1.89)	4.18 [0.9]	(4.39)
Calibrator	98.8 [21.4]	1.19 [0.3]	(1.19)	2.09 [0.5]	(2.09)
Sigma® ethanol control	188.2 [40.9]	2.02 [0.4]	(1.07)	3.20 [0.7]	(1.69)

f. All specific performance characteristics tests were run after normal recommended equipment quality control checks were performed (refer to your Dimension® Operator's Guide).

g. Specimens at each level were analyzed in duplicate for 20 days. The within-run and between-day standard deviations were calculated by the analysis of variance method.

Sigma® is a registered trademark of Sigma Chemical Company, St. Louis, MO 63178.

### Method Comparison

#### Regression Statistics<sup>b</sup>

Comparative Method	Slope	Intercept mg/dL [mmol/L]	Correlation Coefficient	n <sup>i</sup>
ALC method on the aca® discrete clinical analyzer				
Serum	1.01	1.75	0.998	87
Whole blood Supernatant <sup>l</sup>	1.01	1.82	0.999	200
Plasma	0.96	3.04	0.999	99
Urine	1.01	1.53	0.999	92
Abbott TDx® Plasma Gas Chromatography	0.92	2.45	0.997	99
Serum	0.99	1.45	0.997	87
Whole Blood Supernatant <sup>l</sup>	0.34	0.62	0.993	200
Urine	1.00	2.85	0.999	92

h. Model equation for regression statistics is: [results of Dimension® system] = slope x [comparative method results] + intercept.

i. Aliquots of clinical specimens received for ALC analysis.

j. Whole blood samples were analyzed by gas chromatography. Supernatants from whole blood were analyzed by the Dimension® system. Results were not corrected for dilution factor of three (3) to obtain "whole blood" results.

TDx® is a registered trademark of Abbott Diagnostics, Abbott Park, IL 60064.

### Specificity

#### HIL Interference

The ALC method was evaluated for interference from hemolysis, icterus and lipemia according to CLSI/NCCLS EP7-P. Bias, defined as the difference between the control sample (does not contain interferent) and the test sample (contains the interferent), is shown in the table below. Bias exceeding 10% is considered "interference".

Substance Tested	Test Concentration [SI Units]	ALC Concentration [SI Units]	Bias (%) <sup>k</sup>
Hemoglobin (hemolysate)	1000 mg/dL [0.62 mmol/L] (monomer)	93 mg/dL [20.4 mmol/L]	<10
Bilirubin (unconjugated)	20 mg/dL [342 µmol/L]	93 mg/dL [20.4 mmol/L]	<10
Lipemia (Intralipid®)	1000 mg/dL [11.3 mmol/L]	92 mg/dL [20.4 mmol/L]	<10

k. Analyte results should not be corrected based on this bias.

\*Intralipid® is a registered trademark of Fresenius Kabi AG, Bad Homburg, Germany.

### Non-Interfering Substances

The following substances do not interfere with the ALC method when present in serum in the amounts indicated. Systematic inaccuracies (bias) due to these substances are less than 10% at ethyl alcohol concentration of 100 mg/dL [22.0 mmol/L].

Substance	Test Concentration	SI Units
Acetaminophen	0.025 mg/dL	1.66 µmol/L
Acetone	100 mg/dL	17.2 mmol/L
Amikacin	15 mg/dL	256 µmol/L
Ampicillin	5.3 mg/dL	152 µmol/L
Ascorbic Acid	5 mg/dL	227 µmol/L
Caffeine	10 mg/dL	51.5 µmol/L
Carbamazepine	3 mg/dL	127 µmol/L
Chloramphenicol	5 mg/dL	155 µmol/L
Chlordiazepoxide	1 mg/dL	33.3 µmol/L
Chlorpromazine	0.2 ng/dL	6.27 µmol/L
Cholesterol	500 mg/dL	12.9 mmol/L
Cimetidine	10 mg/dL	0.4 mmol/L
Creatinine	30 mg/dL	2652 µmol/L
Dextran 40	6000 mg/dL	1500 µmol/L
Diazepam	0.5 mg/dL	17.6 µmol/L
Erythromycin	6 mg/dL	81.6 µmol/L
Ethanol	400 mg/dL	86.8 mmol/L
Ethosuximide	25 mg/dL	1770 µmol/L
Ethylene Glycol	250 mg/dL	40.3 mmol/L
Furosemide	6 mg/dL	181 µmol/L
Gentamicin	12 µg/dL	25 µmol/L
Heparin	3 U/mL	3000 U/L
Ibuprofen	50 mg/dL	2425 µmol/L
Immunoglobulin G	5 g/dL	50 g/L
Lactic Acid	100 mg/dL	11.1 mmol/L
Lidocaine	1.2 mg/dL	51.2 µmol/L
Lithium	2.3 mg/dL	3.2 mmol/L
Mannitol	500 mg/dL	27.4 mmol/L
Methanol	100 mg/dL	31.2 mmol/L
Nicotine	2 mg/dL	0.1 mmol/L
Penicillin G	25 U/mL	25000 U/L
Pentobarbital	8 mg/dL	354 µmol/L
Phenobarbital	10 mg/dL	421 µmol/L
Phenytoin	5 mg/dL	198 µmol/L
Primidone	4 mg/dL	183 µmol/L
Propoxyphene	0.2 mg/dL	4.91 µmol/L
Protein: Albumin	6 g/dL	60 g/L
Protein: Total	12 g/dL	120 g/L
Salicylic Acid	60 mg/dL	4.34 mmol/L
Theophylline	4 mg/dL	222 µmol/L
Triglycerides	3000 mg/dL	33.9 mmol/L
Urea	500 mg/dL	83.3 mmol/L
Uric Acid	20 mg/dL	1190 µmol/L
Valproic Acid	50 mg/dL	3467 µmol/L

### Analytical Sensitivity: 1 mg/dL [0.2 mmol/L]

The analytical sensitivity represents the lowest concentration of ethyl alcohol that can be distinguished from zero. This sensitivity is defined as the mean value (n = 20) plus two standard deviations of Level 1 ALC calibrator.

**Symbols Key:** See adjacent panel.

Dimension® and Flex® are trademarks of Siemens Healthcare Diagnostics Inc.

©2008 Siemens Healthcare Diagnostics Inc.

All rights reserved

## Dimension® clinical chemistry system

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**Bibliography/Literatur/Bibliographie/Bibliografia/Bibliografía:**

1. Bonnichsen R and Lundgren G. Comparison of the ADH and the Widmark procedures in Forensic Chemistry for Determining Alcohol, Acta. Pharmacol et Toxicol 13, 256-266 (1957).
2. Bergmeyer HU. Methods of Enzymatic Analysis, Academic Press, New York, NY. 1965, p 285.
3. Tietz NW. Textbook of Clinical Chemistry, W.B. Saunders Co., Philadelphia, PA 1986, pp 52-53 (techniques and procedures to minimize laboratory infections) and p 1706 (specimen collection and storage recommendations).
4. Henry JB. Clinical Diagnosis and Management by Laboratory Methods, W.B.Saunders Co., Philadelphia, PA 1979, p 507.
5. Tietz NW. Clinical Guide to Laboratory Tests, W.B. Saunders Co., Philadelphia, PA 1990, p 207.
6. Jacobs DS. et al, Laboratory Test Handbook with DRG Index, Mosby/Lexi-Comp Inc., Stow, OH. 1984, p 525.
7. Clinical and Laboratory Standards Institute/NCCLS. *Tubes and Additives for Venous Blood Specimen Collection; Approved Standard – Fifth Edition*. CLSI/NCCLS H1-A5 (ISBN 1-56238-519-4). CLSI, 940 West Valley Road, Suite 1400, Wayne, PA 19087-1898 USA, 2003.
8. Windholz M. et al, The Merck Index, An Encyclopedia of Chemicals and Drugs, Merck & Co., Inc., Rahway, NJ. 1983, p 1376.
9. Hawks RL, Chiang CN. Urine Testing for Drug Abuse. NIDA Research Monograph 73, 1986, pp. 103-104.
10. Dubowski KM. Alcohol Analysis: Clinical Laboratory Aspects, Part 1. Lab Mgt. vol 20(3). Mar 1982 pp 43-54.
11. Blanke RV. et al, Drug Concentrations & Driving Impairment. JAMA vol 254, Nov. 8, 1985 pp. 2618-2621.
12. Sunshine I. Methodology for Analytical Toxicology, CRC Press, Inc, Cleveland, OH, 1975 pp 152-153.

**Symbols Key**  
**Symbolschlüssel**  
**Explication des Symboles**  
**Interpretazione simboli**  
**Clave de los Símbolos**

	Do not reuse / Nicht zur Wiederverwendung / Ne pas réutiliser / Non riutilizzare / No reutilizar
<b>EXP</b> <small>CCYY-MM-DD</small>	Use By / Verwendbar bis / Utiliser jusque / Utilizzare entro / Fecha de caducidad
<b>LOT</b>	Batch Code / Chargenbezeichnung / Code du lot / Codice del lotto / Código de lote
<b>REF</b>	Catalogue Number / Bestellnummer / Référence du catalogue / Numero di catalogo / Número de catálogo
	Caution, consult accompanying documents / Achtung, Begleitdokumente beachten / Attention voir notice d'instructions / Attenzione, vedere le istruzioni per l'uso / Atención, ver instrucciones de uso
	Manufacturer / Hersteller / Fabricant / Fabbricante / Fabricante
<b>EC REP</b>	Authorized Representative in the European Community / Bevollmächtigter in der Europäischen Gemeinschaft / Mandataire dans la Communauté européenne / Mandatario nella Comunità Europea / Representante autorizado en la Comunidad Europea
	Contains sufficient for <n> tests / Inhalt ausreichend für <n> Tests / Contenu suffisant pour "n" tests / Contenido suficiente para "n" saggi / Contenido suficiente para <n> ensayos
<b>IVD</b>	In Vitro Diagnostic Medical Device / In-Vitro-Diagnostikum / Dispositif médical de diagnostic in vitro / Dispositivo medico-diagnostico in vitro / Producto sanitario para diagnóstico in vitro
	Temperature Limitation / Temperaturbegrenzung / Limites de température / Limiti di temperatura / Limite de temperatura
	Consult Instructions for Use / Gebrauchsanweisung beachten / Consulter les instructions d'utilisation / Consultare le istruzioni per l'uso / Consulte las instrucciones de uso
	Non-sterile / Nicht steril / Non stérile / Non sterile / No estéril
<b>CE</b>	CE Mark / CE Zeichen / Marquage CE / Marchio CE / Marca CE
<b>CONTENTS</b>	Contents / Inhalt / Contenu / Contenuto / Contenido
	Reconstitution Volume / Rekonstitutionsvolumen / Volume de reconstitution / Volume di ricostituzione / Volumen de reconstitución
<b>LEVEL</b>	Level / Konzentration / Niveau / Livello / Nivel

2006-06\_EFIGS

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Dimension® y Flex® son marcas comerciales de Siemens Healthcare Diagnostics Inc.